PFLEGEROLLSTUHL ROM

Rome Care Wheelchair

22153 | 22154 | 22156

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NOTIZEN

Dear User,

Thank you for choosing this product. This user manual describes how to prepare, operate, use, maintain and care for the product. It is a guideline for users and caregivers to ensure the safe and risk-free use of the product. Please read this user manual carefully and keep it readily at hand at all times.


CONTENTS

The product is supplied in a cardboard box. The cardboard box serves for the future storage of the product and must not be discarded. Please first check whether the contents of the box is complete and undamaged. Notify the supplier immediately upon receipt should you detect any damage. Complaints raised at a later point in time cannot be accepted.

Package content

- Storage box
- Rome care wheelchair with leg rest
- Maintenance manual
- This manual

TECHNICAL SPECIFICATIONS

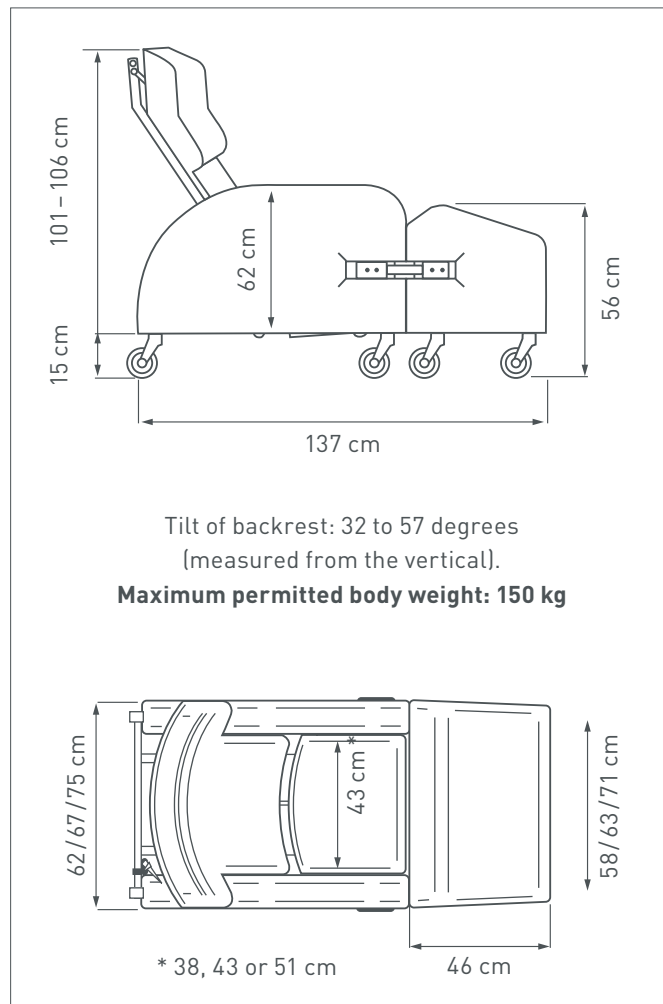Dimensions

Width (SW 38/43/51)	62/67/75 cm
Width of leg rest (SW 38/43/51)	62/67/75 cm
Length	88 cm
Length of leg rest	46 cm
Total length with leg rest	137 cm
Total height	116 to 121 cm
Height of leg rest	56 cm
Height of sliding handle	10 cm
Ground clearance	15 cm
Sitting angle	20° to 35°
Seat depth	43/47/50 cm
Seat width	38/43/51 cm
Seat height	48.5 cm
Backrest angle	32° to 57°
Backrest height	74 to 80 cm
Lower leg length	47 cm
Height of armrest to sitting surface	13,5 cm
Length of armrest (from the backrest)	60 to 75 cm depending on position of backrest
Width of armrest cushions	12 cm
Diameter of rollers	9 cm
User weight	150 kg
Tare weight (SW 38/43/51)	40/41/42 kg
Weight of leg rest	11 kg

Materials

Frame	Steel, corrosion-resistant, powder-coated
Mounting parts	Plastic (100% PVC)
Top cover	Artificial leather (100% vinyl)
Upholstery	Foam (100% PU), flame-retardant acc. to BS 5852

ROME CARE WHEELCHAIR

OVERVIEW

- 1 Sliding handle
- 2 Lever for backrest
- 3 Backrest
- 4 Armrest
- 5 Leg rest
- 6 Mounting clip
- 7 Smooth castors
- 8 Smooth castors with locking brake

Options

Art. no. 22153 = seat width 43 cm

Art. no. 22154 = seat width 51 cm

Art. no. 22156 = seat width 38 cm

Accessories

Art. no. Z-22153-13 = footplate for Rome care wheelchair

Type plate

The type plate is attached to the frame or beneath the seat of the care wheelchair.

CAUTION!

The signs on the care wheelchair must be legible! Illegible or missing signs must be replaced immediately; please contact CARELINE to arrange this. The type plate must not be removed and includes the serial number which is required for precise identification of the product.

INTENDED PURPOSE

The Rome care wheelchair is designed as a sitting aid for indoors, and its use allows patients to take part in various activities of everyday life. Furthermore, the Rome care wheelchair allows a stable and upright sitting position to be achieved. This guided sitting position helps to relieve involuntary dyskinesias caused by the disease. The Rome care wheelchair

is available in seat widths of 38 cm, 43 cm and 51 cm. The maximum user weight is 150 kg.

SAFETY INSTRUCTIONS

Read the user manual for use in full before initial use. The care wheelchair must not be set up or operated without first reading the user manual for use. Familiarise yourself with the care wheelchair before initial use. Make sure that there are no objects or obstacles in the chair's field of action that could cause injury to the patient, their relative or medical/nursing staff.

The safety of the patient took priority when developing the Rome care wheelchair. Nevertheless, you should watch out for possible pinch points for users and operators when using and adjusting the care wheelchair. The brake rollers must be locked when using the care wheel chair as a sitting aid. It must be ensured that an attendant is in the same room. The care wheelchair should preferably be used on a level, smooth surface. Soft carpets make it more difficult to push the care wheelchair. Unevenness and slopes must be avoided, as these pose an elevated risk of accidents. Never place the care wheelchair on steep ground. The care wheelchair is only suitable as a sitting aid for one person. Before each use, make sure that the care wheelchair is undamaged, all adjustable parts and fixed properly and the brake rollers are functioning.

Please note that upholstered parts exposed to direct sunlight may heat up and cause skin injury if touched. Therefore cover these parts or protect the care wheelchair from sunlight. Avoid unbraked pushing against an obstacle (step, ledge). There is a risk of injury. Do not use any high-pressure cleaners or sharp, corrosive chemicals for cleaning.

The product has been tested in accordance with standard BS 5852 1 and 2 and is therefore considered of low flammability. You should still protect cushions and add-on parts from ignition sources such as cigarettes, lighters and open flames.

If you notice a defect or unusual behaviour in your care wheelchair, it should be discontinued from use immediately. For repair and replacement, contact CARELINE.

Please note: "CAUTION!" marks especially safety-relevant information, so this must be observed!

INDICATION

- Huntington's disease
- Restriction of sitting ability
- Hyperkinesia or hypokinesia
- Other neurological diseases such as stroke

CONTRAINDICATION

The use of the care wheelchair is not approved for those with

- unstable circulation
- sudden dizziness
- patient discomfort while sitting in the Rome care wheelchair

USE

Leg rest

The leg rest consists of a stable wooden frame, which is securely padded with foam. The leg rest has been perfectly designed for the Rome care wheelchair. The internally offset rear wheels enable easy manoeuvring in a coupled state. The durable rubber clasp fixes the leg rest securely on the care wheelchair, even under extreme loads. To lock the leg rest into place on the care wheelchair, hook in the clasp in the bracket on the care wheelchair and pull the lever forwards. Repeat the process on the other side. To release the bracket, pull the lever backwards.

CAUTION!

There is a risk of pinching, especially with moving parts!

Backrest angle

Operate the lever for the backrest angle and push the sliding bar upwards or downwards to adjust the backrest angle. When adjusting the backrests, the sitting angle will adapt to ensure an anatomically correct sitting position.

Castors with locking brake

Locking brakes are attached to the rear castors of the care wheelchair to prevent the care wheelchair from rolling away. For this, the locking lever on the castor is pushed downwards with the toes. To release the brake again, the foot level must be pushed back upwards with the toes.

CAUTION!

The functioning of the locking brakes must be checked regularly!

Lifting points

Due to its heavy weight, the care wheelchair should always be lifted by two people. You can lift the care wheelchair safely from the seat frame. Lifting from add-on parts such as side parts, backrest or the wheels is not permitted.

PRODUCT TRAINING

If necessary, the CARELINE field service can perform product training for the Rome care wheelchair.

CLEANING AND DISINFECTION

All parts of the Rome care wheelchair should be cleaned and disinfected regularly. To clean, use a damp cloth. In case of more severe soiling, use a mild cleaning agent. The wheels can be cleaned using a damp brush with plastic bristles. Use a commercial surface disinfectant for disinfection; observe the manufacturer's instructions for use in this regard.

CAUTION!

Do not use any high-pressure or steam cleaners!

MAINTENANCE BY CARELINE

Any maintenance and repair work performed on the Rome care wheelchair must be exclusively performed by CARELINE in accordance with the maintenance contract. Improper performance of maintenance/repairs will void all liability and warranty claims!

CAUTION!

The screw connections must be checked again immediately after repairs!

INSPECTION BY THE USER

Daily inspection

Travel behaviour

Check that the brakes are functioning before use. If the brake is activated, it should not be possible to push the care wheelchair away. If the brake is released, the care wheelchair should

ROME CARE WHEELCHAIR

be easy to push without any grinding noises. If you notice any unusual braking behaviour, notify CARELINE immediately and discontinue use of the care wheelchair.

When moving the care wheelchair, pay attention to normal travel behaviour. If you notice that the travel behaviour has changed or that the care wheelchair is becoming more unstable, stop using the care wheelchair and contact CARELINE.

Defects and contamination

Check that the care wheelchair including seat and back cover, add-on parts, frame and wheels are clean and undamaged. Contact CARELINE if you discover any damage.

Monthly inspection

Backrest adjustment and moving parts

Check all moving parts are moving smoothly and functioning with little noise. Squeaking noises indicate damage and should be checked by CARELINE.

Screw and plug-in connections

Check all connections are undamaged, rust-free and sitting firmly.

RE-USE

The Rome care wheelchair can be re-used after inspection by CARELINE, following replacement of any damaged parts and disinfection.

USEFUL LIFE

The expected useful life of the Rome care wheelchair is five years if used in accordance with this manual and undamaged. It can be used beyond this period if it is in a safe condition and does not show any signs of damage.

WARRANTY

The statutory warranty period of two years from date of purchase applies. The warranty covers any defects of the product which are demonstrably due to material or manufacturing defects. Any damage incurred by natural wear and tear, intent, negligence or improper operation or use is excluded from the warranty. The warranty also expires if unsuitable detergents, lubricating oils or greases are used. In the event of non-compliance with the instructions contained in this user manual, improper maintenance or unauthorised modifications to the product without the manufacturer's prior written consent, all liability and warranty claims will expire with immediate effect.

DISPOSAL

Please dispose of the Rome care wheelchair in an environmentally friendly manner via an approved waste disposal company or your local waste disposal facility. Please make sure to comply with any applicable regulations.

NOTES

22153 | 22154 | 22156

CARELINE GmbH

Ottostraße 11 | 41540 Dormagen
T +49 2133 2223 0 | F +49 2133 2223 350
info@careline.de | www.careline.de

CARELINE B.V.

Oude Liermolenweg 8 | 2678 MN De Lier
T +31 88 5012400 | F +31 88 5012450
info@carelinebv.nl | www.carelinebv.nl

CARELINE SAS

29 Allée Eugène Ducretet | 26000 Valence
T +33 4 69118940 | F +33 4 69118949
info@careline-group.fr | www.careline-group.fr