CARELAUF

GEHWAGEN

PATIENT WALKER | LOOPREK

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NOTIZEN


Dear User,

Thank you for choosing this product. If you use your product in accordance with this manual, it will last you many years, making your everyday life easier. Please read this user manual carefully and keep it in a safe place for future reference. Most importantly, please observe the safety instructions.

CONTENTS

The patient walker is supplied in a cardboard box. The cardboard box serves for the future storage of the walker and must not be discarded. Please first check whether the contents of the box is complete and undamaged. Notify us immediately upon receipt should you detect any damage. Complaints raised at a later point in time cannot be accepted.

- Storage box
- Armrest (A)
- Two adjustable beams (B)
- Chassis (C)
- Connectors: 8 hexagon socket screws, 2 adjustment screws to adjust height of walker, tools for assembly and maintenance
- This manual

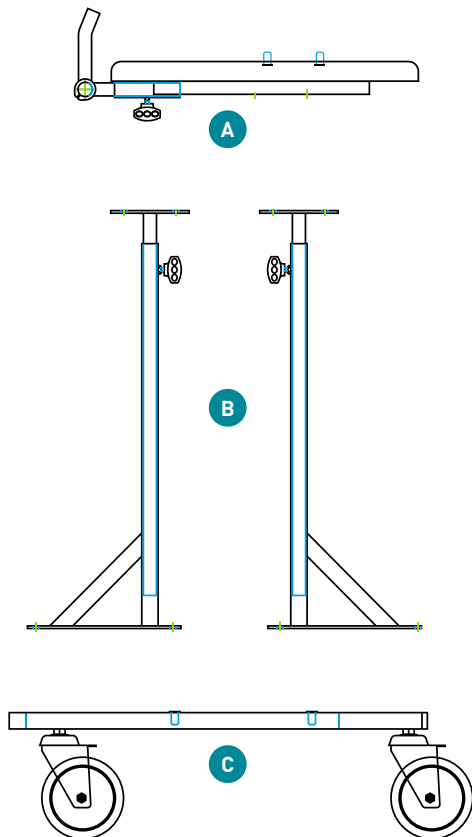**INTENDED PURPOSE**

This patient walker is intended only for patients with mobility impairments. It helps the patient to remain stable and relieves strain on the legs. The patient walker is only suitable for indoor use.

PLEASE NOTE


The attending physician should decide whether this walker is suitable for the patient, taking into account the patient's state of health.

SAFETY INSTRUCTIONS

Do not use the patient walker on surfaces with a slope steeper than 3.5° , on ramps, or in stairwells. Do not use the patient walker for transporting individuals, loads etc. Maximum load: 130 kg.

ASSEMBLY

Remove all parts from the cardboard box and lay them out on the floor, paying careful attention to not damaging the surface.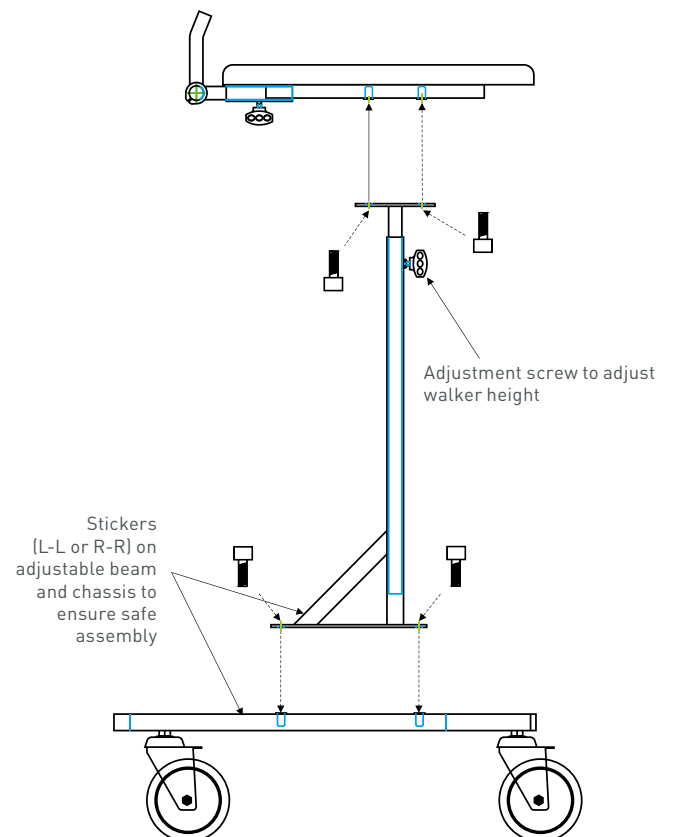

Position the chassis (C) so that it rests on the wheels and activate the rear wheel brakes. Mount the adjustable support beams (B) on the chassis and tighten screws. Bear in mind

that there is a right and left beam. A label on each beam indicates whether it is the left ("L") or right ("R") beam. Each beam must be mounted to the respective side of the chassis. Make sure that the reinforcement of the adjustable beam is mounted facing towards the front of the chassis (in the direction of movement).

Attention: We cannot guarantee the stability of the patient walker in the event of incorrect assembly!

Use the screws supplied to hold the adjustable beams in place and tighten with wrench. The patient walker should be assembled by two people. Then, lower the armrest onto the adjustable beams. Use the screws provided to mount it firmly and tighten with wrench. You can adjust the height of the armrest with the adjustment screws.

ATTENTION

The armrest should always be in a horizontal position and firmly attached. Should the armrest tilt or should the wheels of the chassis not rest firmly on the ground, do not use the patient walker and check the armrest height settings.

MATERIAL

The patient walker's frame is made of powder-coated square steel tubes. The adjustable beams and handles are made of stainless steel. The u-shaped armrest is made of wooden board, padded on both sides. Padding seams are high-frequency welded.

TECHNICAL SPECIFICATIONS

- Dimensions: W 67 x H 84–131 x D 75 cm
- Inside width between armrests: 40 cm
- Wheels: 4 x 12,5 cm, 2 two of which with brakes
- Weight: 11,5 kg
- Weight including packaging: 13,5 kg
- Max. load/supported weight: 130 kg**


USE

- Before each use, please ensure that:
- All wheels are firmly attached to the chassis, are resting stably on the ground, can move freely, and are not worn
 - All screws are firmly tightened
 - All adjustment screws are tightly locked
 - The handles are firmly screwed in place
 - The walker frame is free of damage

Should any of the above points not apply, do not use the patient walker. Defective parts must be replaced, loose screws must be tightened, and the adjustment screws checked before

each use. Only use original spare parts in the event of repairs, as the load-bearing capacity and safety of the patient walker can otherwise not be guaranteed.

Adjust armrest height by loosening the adjustment screws (1) on the left and right adjustable beam, bring the armrest to the desired height, and tighten the screws again firmly. Use adjustment screws (2) to adjust the distance between handles and armrest. If required, loosen screws (3) to adjust the inclination of the handles. After every adjustment, make sure that the armrest is in a horizontal position.

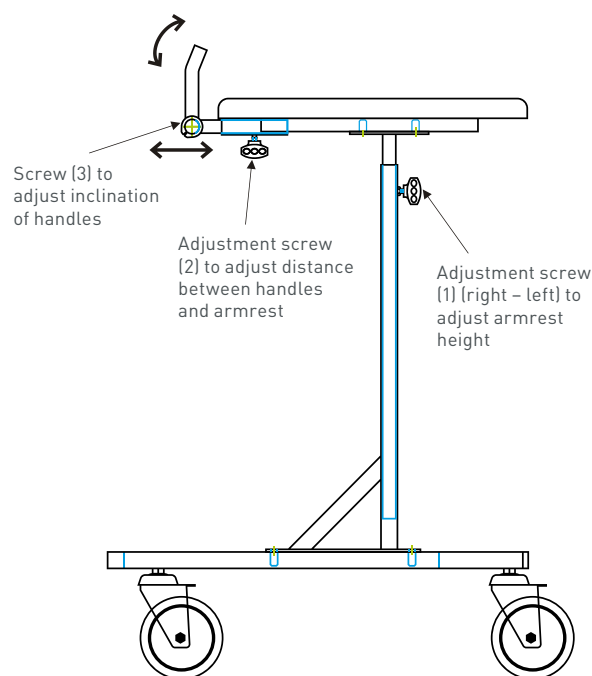Position the patient in the walker so that they can support themselves by resting their lower arm on the armrest, and the patient's chest is positioned within the u-shaped opening between the armrests when the patient is holding onto the handles with their hands.

Attention: Never adjust the height of the patient walker while a patient is using it; risk of injury!

GUARANTEE

We grant 2 years warranty on manufacturing or material defects. Effects of improper use, negligence, poor maintenance, or poor storage conditions are not covered. If repairs or modifications of the patient walker or its parts are carried out by an unauthorised person, this voids the warranty. Any product liability on the part of the manufacturer is precluded in such events.

Furthermore, the manufacturer cannot accept liability for any damage to individuals, property, or animals in the following cases:

- Improper use of the device
- Use by unfit individuals
- Incorrect assembly of parts or accessories
- Modification or alteration without the manufacturer's prior consent
- Using non-original spare parts or accessories
- Disregarding the instructions provided in this user and assembly manual
- Extraordinary events

CLEANING AND DISINFECTION

All components of the patient walker should be cleaned and disinfected regularly. Use a damp cloth for cleaning and a mild detergent to remove any heavier soiling. Use a commercially available surface disinfectant in accordance with the manufacturer's instructions. Please do not use aggressive or abrasive cleaning agents that could cause mechanical or chemical damage to the frame's surface or the padded armrest.

INFORMATION ON DISPOSAL

Please dispose of the patient walker in an environmentally friendly manner via an approved waste disposal company or your local waste disposal facility. Please make sure to comply with any applicable regulations.

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CARELINE GmbH & Co. KG
CARELINE B.V.